

FREE SPIRIT ® KNEE and LEG WALKER

FOLDING INSTRUCTIONS



The tiller (handle bar) is shipped unassembled. Slip the tiller over the front bracket. Insert bolts through bracket and tighten nuts. Slip the padded knee rest into the hold provided. Lock it in place at the desired height, using the black knob and bolt provided. Tighten the knob securely.

CAUTION: When unit is in use, make sure black pull pin on tiller latch is in the closed position and quick release lever is folded in toward unit.

TO FOLD THE STEERING COLUMN (MAST) FOR EASY TRANSPORT:



Step 1



Step 2



Step 3



Step 4



Step 5

Scott About, LLC
P. O. Box 886
Helotes, Texas 78023-0886
Phone/Fax: 210-695-1136
<http://scootaboutkneescooters.com>

To Lower Mast:

Step 1 – open tiller lever

Step 2 – loosen silver screw knob opposite the tiller lever, a couple of turns

Step 3 – pull out black knob, allowing the mast to be lowered

Step 4 – move mast slightly out to the side, tuck it back under the knee rest platform

Note: If the knee platform is in the lowest position, the mast will not tuck completely under the knee platform.

To Raise Mast:

Reverse Steps 1 - 4 above

Step 5 – Silver bolt should be tightened enough so that mast is firm, not wobbling. The silver lever should close to ¼" – ½" from casing (not touching it) as shown in Step 5 above.